
# ELSODENT

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Thermoplastic plate for hard relining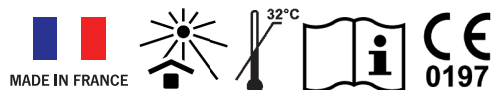

## TopBase

G-PHARMA - 35, avenue du Gros Chêne - B.P. 10279 HERBLAY  
95617 CERGY-PONTOISE Cedex FRANCE  
elsodent@elsodent.com - www.elsodent.com

TopBase GB

### Description & indication

TopBase is a thermoplastic plate for semi-definitive hard-relining of dentures made of resin.

### Safety notes

#### Warnings

• TopBase is an inert and stable plastic plate. In the form of a plate, this compound does not have any known allergy or sensitization. However, as a precaution avoid contact with the eyes to prevent irritation and potential damage to the cornea. In case of contact with eyes, rinse immediately with plenty of water. See a doctor.

#### Precautions

• Wear goggles, a mask, protective clothing and gloves.

### Contraindications

TopBase is contraindicated in patients with a history of allergic reaction to the product.

### Instruction for use

#### Preparation of the prosthesis to rebase

1. Clean the intrados of the prosthesis, rinse and dry.
2. Make the surface slightly rough, with sandpaper, a bur or sandblast the intrados of the prosthesis with alumina oxide. [cf 1]
3. Clean the surface with alcohol and dry thoroughly with dry air free of water and oil.

### Relining

1. Place the plate on the intrados of the prosthesis, taking care to centre it. [cf 2a - 2b]
2. Using a hair dryer, or a heat air gun (temperature at 250°C), heat the plate starting at the centre, then run the hot air source on the sides to distribute the heat evenly. [cf 3]
3. Once the plate has softened uniformly, it begins to become transparent and adheres directly to the intrados of the prosthesis. [cf 4a - 4b]
4. If necessary, with your gloves slightly wet, push the soften plate into the intrados to get a better contact, then and an excellent adhesion, and to avoid trapping air bubbles. [cf 5]
5. Start immediately to remove material in excess, on the edges of the prosthesis with your fingers wets. [cf 6-7]
6. Immerse TopBase in hot water, at 70°C [cf 8], before putting it in mouth, to avoid any risk for the plate to stick to the mucosa, in case of hyposalia.
7. Once in mouth, ask the patient to bite firmly for 2 minutes, in order to accurately register all the details of the surface of the mucosa. [cf 9]
8. Remove from the mouth. If a modification is needed, TopBase can be re-heated indefinitely. Then, put the denture back into the mouth. Don't forget to immerse it into hot water for 15 seconds.
9. Remove excesses and polish with rubber cup or disc. You can also re-heat the edges to smooth them.

### Advice

The contact and therefore the retention in mouth will be improved if the prosthesis is previously wet before putting it into the mouth. Always immerse the denture in the bowl of hot water, each time before putting it back in the mouth.

### Setting time

The total setting time until complete crosslinking is 3 minutes maximum.

### Storage and shelf-life

Do not expose the product to sun-light or warm sources. Store at a temperature not exceeding 32°C. When stored under such conditions, TopBase plates have a shelf-life of 36 months.

### Presentation & references

TPU - Kit of 2 maxillary plates

TPD - Kit of 2 mandibular plates

TPK - Kit of 1 mandibular plate & 1 maxillary plate

TPCLU - Clinical kit of 10 maxillary plates

TPCLD - Clinical kit of 10 mandibular plates

### Important notice

Do not throw away the box and the instruction for use until the product is finished. High humidity could damage the packaging on which important instructions and legal information are printed. Storage at a too high temperature could cause to age prematurely.

### Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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